NEW!

Symphony PLUS Pumping Programs

For mothers of term and preterm infants.

Quick start instructions

The Symphony PLUS breast pump, with its research-based pumping programs, has been specifically developed to support mothers throughout their lactation journey, helping to initiate, build and maintain an adequate milk production.

Double-pumping is recommended, as this can help to increase and maintain milk production.

Symphony PLUS contains two pumping programs:


1 The **INITIATE** program supports mothers to successfully initiate milk production. The program mimics the irregular and more rapid sucking and pausing pattern of a new born infant in the first days of lactation.

2 The **MAINTAIN** program is designed to build and maintain an adequate milk production and optimize milk output after secretory activation (milk coming in) has occurred. The program, with its 2-Phase Expression technology, mimics an infant's sucking pattern during established lactation.

1 **The INITIATE program**
Use the INITIATE program directly after birth:
✓ Every 2-3 hours or 8 times a day
✓ For up to five days, **OR**
✓ Until you have expressed a minimum of 20 ml from both breasts combined in 3 consecutive pumping sessions.
Once selected, the INITIATE program runs automatically for 15 minutes, with several pauses. It is important to complete the full program. At the end of the program the display shows "Program complete". The pump switches off automatically.

2 **The MAINTAIN program**
Use the MAINTAIN program after secretory activation (milk coming in) if:
✓ The INITIATE Program has been successfully used as per above, **OR**
✓ Breastfeeding has been established, **OR**
✓ You are more than five days post birth.
Continue pumping until the breasts feel well drained and soft all over. Switch the pump off manually when the pumping session is finished. On average, pumping sessions last approximately 15 minutes.

 Note: These quick instructions do not replace the regular Instructions for Use. Please refer to Symphony Instructions for Use for more detailed information.

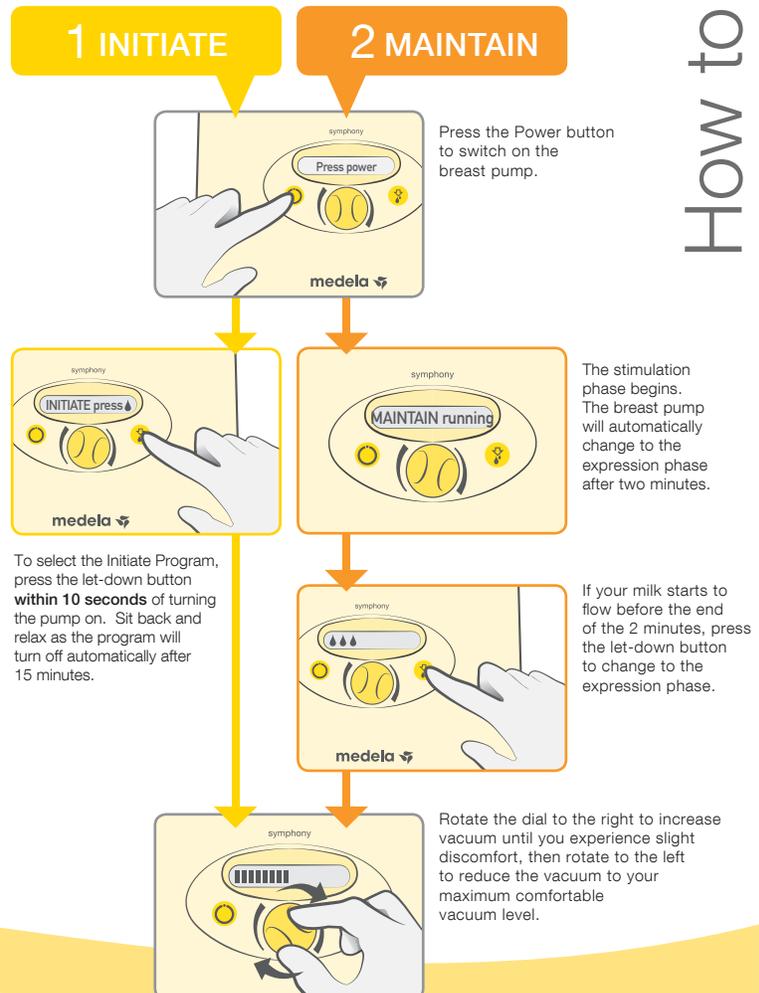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Fabriqué par Medela AG, Lättichstrasse 4b, 6341 Baar, Suisse.
Distribué par Medela Canada Inc. 4160 Sladeview Crescent, Unité #8, Mississauga, Ontario, L5L 0A1